

*Scan for more information*



# **Activon<sup>®</sup> Tube**

***100% Medical Grade Manuka honey***

*Instructions for use*



Advancis Medical, Lowmoor Business Park,  
Kirkby-in-Ashfield, Nottingham, NG17 7JZ, England

**Tel:** +44 (0) 1623 751500

**Fax:** +44 (0) 871 264 8238

**Email:** [info@advancis.co.uk](mailto:info@advancis.co.uk)

**Web:** [www.advancis.co.uk](http://www.advancis.co.uk)



# Activon® Tube

100% Medical Grade Manuka honey

EN

## Product Description

Activon® Tube contains 100% Medical Grade Manuka honey with no additives. This wound filler will eliminate odors and provides a moist wound healing environment that facilitates autolytic debridement. Activon® Tube can be used as a wound filler for use in cavities.

## Indications

Use on Activon® honey impregnated dressings.

OTC Use: For the management of minor wounds, minor ulcerations, minor burns, minor abraded skin and minor irritated areas.

Prescription Use: Under the supervision of a healthcare professional, use for the management of diabetic foot ulcers; leg ulcers (venous stasis ulcers, arterial ulcers and leg ulcers of mixed etiology); pressure ulcers/sores (partial and full thickness); 1st and 2nd degree partial thickness burns; grafted and donor sites and traumatic surgical wounds.

## Contra-indications

Known allergy to bee-venom.

## Directions for use

Twist off cap, apply the wound filler as a top up to an Activon® honey impregnated wound dressing, immediately prior to application to increase concentration of honey. Apply dressing in accordance with 'Instructions for Use' supplied with dressing. Single patient use only. Can be used for up to 90 days after opening.

## Disposal


Dispose as clinical waste.

## Storage

Store at room temperature. Our honey is 100% Manuka honey. As a natural product this can vary in color and in exceptional circumstances crystallization can occur.

## Warnings and Precautions

Although no known instance of increased levels of blood sugar in patients with diabetes it is advisable to closely monitor the levels. Increased pain may be experienced due to osmotic action.

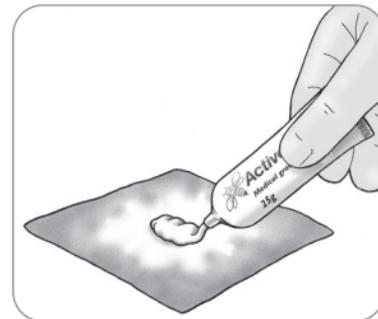

<b>REF</b>	See Packaging	Catalogue #
		Do not use if package is damaged
<b>STERILE R</b>		Sterilized using irradiation
		Do not re-sterilize
		Caution, consult instructions for use
<b>CE</b> <b>1639</b>		CE Mark
		Use within 90 days of opening