

Scan for more information



Eclipse[®] Adherent

***Super absorbent dressing with soft
silicone contact layer***

Instructions for use



Advancis Medical, Lowmoor Business Park,
Kirkby-in-Ashfield, Nottingham, NG17 7JZ, England


Tel: +44 (0) 1623 751500

Fax: +44 (0) 871 264 8238

Email: info@advancis.co.uk

Web: www.advancis.co.uk



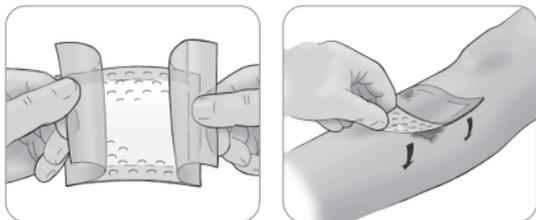

Eclipse® Adherent

EN

Superabsorbent dressing with Silfix® soft silicone contact layer

Product Description

Eclipse® Adherent is an exudate management wound dressing. The dressing combines a Silfix® soft silicone contact layer, a rapid wicking layer, a CrystaLock™ layer, which is highly absorbent and locks away exudate, a sub-backing layer for even exudate distribution and a non-striethrough backing to prevent leakage.

Indications

Moderate to heavily exuding wounds such as leg ulcers, pressure ulcers, diabetic foot ulcers, sloughy wounds, granulating wounds, lymphatic legs, post-operative and dehisced wounds, fungating wounds and donor sites.

Contra-Indications

Arterial bleeds and heavily bleeding wounds.

Directions for Use

Apply to wound (white silicone face down, beige facing up) allowing a minimum of a 2cm border overlap around the wound area. Do not cut Eclipse® Adherent. Dressings can be placed side-by-side to cover large wounds. Secure the dressing pad with tape or bandage appropriately. Eclipse® Adherent may be used under compression bandages. Eclipse® Adherent can be used as a primary or secondary dressing.

Frequency of change

Wear time will depend on the level of exudate. Eclipse® Adherent must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to 7 days.

Disposal

Dispose as clinical waste.

REF	See Packaging	Catalogue #
		Single patient use
		Product is not made with natural rubber latex
		Do not use if package is damaged
STERILE EO		Sterilized using ethylene oxide
		Do not re-sterilize
		Consult instructions for use
		Keep dry
		Keep away from sunlight
		Manufacturer