

Scan for more information



Eclipse[®]

Super absorbent dressing

Instructions for use



Advancis Medical, Lowmoor Business Park,
Kirkby-in-Ashfield, Nottingham, NG17 7JZ, England

Tel: +44 (0) 1623 751500

Fax: +44 (0) 871 264 8238

Email: info@advancis.co.uk

Web: www.advancis.co.uk



Eclipse®

Superabsorbent dressing

Product description

Eclipse® is an exudate management wound dressing. The dressing combines a rapid wicking layer, a CrystalLock™ layer, which is highly absorbent and locks away exudate, a sub-backing layer for even exudate distribution and a non-striktthrough backing to prevent leakage.

Indications

Moderate to heavily exuding wounds such as leg ulcers, pressure ulcers, diabetic foot ulcers, sloughy wounds, granulating wounds, lymphatic legs, post-operative and dehiscid wounds, fungating wounds and donor sites.

Contra-indications

Arterial bleeds and heavily bleeding wounds.

Directions for use



Eclipse® is placed white face down on the wound surface, with beige backing uppermost. For larger wounds place several dressings side by side. Protect friable skin, or drier areas of the wound with a non-adherent dressing under Eclipse®. Do not cut Eclipse® dressings. Secure the dressing pad with tape or bandage appropriately. Eclipse® may be used under compression bandages. Eclipse® can be used as a primary or secondary dressing.

Frequency of change

Wear time will depend on the level of exudate. Eclipse® must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to 7 days.

Disposal

Dispose as clinical waste.

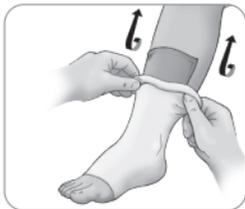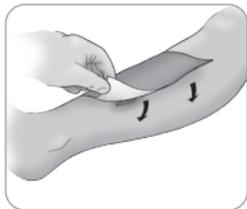

REF See Packaging

Catalogue #



Single patient use



Product is not made with natural rubber latex



Do not use if package is damaged

STERILE EO

Sterilized using ethylene oxide



Do not re-sterilize



Consult instructions for use



Keep dry



Keep away from sunlight



Manufacturer